

ASEAN DNA Center

DNA SAMPLE COLLECTION MANUAL (BUCCAL SWAB)

> ASEAN DNA kit includes the following elements:


- 4 sets of sterile cotton swabs (2 in each packet)
- Laboratory request form (the original for the laboratory, copy for the client)
- Disposable gloves
- Stamped addressed envelope

> Preparation for the test:

Please do not eat, drink, smoke and chew gum 1-2 hours before the test. In case of small children 30minutes from the feeding is acceptable.

SWAB COLLECTION

- 1** Put the enclosed gloves on, then take one of the cotton swabs out of the paper packet holding the end without cotton.
- 2** Rub smoothly the inside of one cheek 10 times back and forth. After you finish, put the cotton swab in the paper packet.
- 3** Now take out the other cotton swab and repeat the same action collecting swab from the other cheek.
- 4** Fill in the enclosed laboratory request form and sign all the blanks marked in yellow.

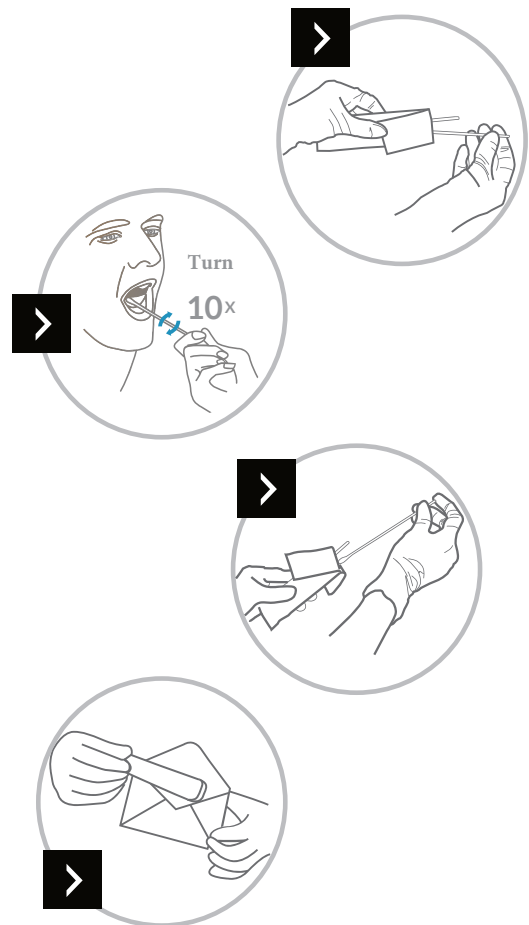

In case of questions feel free to call: +959777438161